Specialized Equipment
for Specialty Healthcare
Medical Technology Industries

464

& 463 user manual

SCAN THE QR CODE TO THE LEFT
TO ACCESS SETUP AND OPERATION
VIDEOS AND RESOURCES FOR THE
464 AND 463 SERIES CHAIRS

Table of Contents

APPLICABLE MODEL(S).....	IV
REVISION HISTORY TABLE	IV
GENERAL INFORMATION	1
Purpose of this Manual.....	1
Intended Use.....	1
Training Options.....	1
Symbols and Warnings.....	2
Safety Instructions	4
Electromagnetic Interference	5
Electromagnetic Compatibility (EMC).....	5
Electrical Requirements	8
Contraindications	9
Transportation and Storage Conditions	9
Disposal of Equipment	9
EQUIPMENT FEATURES	10
INSTALLATION	11
Required Tools	11
Explanation of Instructions	11
Unpacking and Set-up.....	12
Operating Conditions	13
SmartTech™	14
Smart Controller™ Technology.....	14
ComfortSync™	14
User Program Mode.....	14
Hibernation State.....	15
Smart Safety™ Technology.....	15
Safety Lockout.....	15
Shutdown.....	15
Controls.....	16
Headrest Operation	18
Armrest Operation	19
Footrest Operation.....	19
Swivel Base Operation	20
LIST OF ACCESSORIES.....	21
MAINTENANCE	22
Disinfection	22
Cleaning	23
Preventative Maintenance.....	23
Power Cord Removal Instructions.....	23
Fuse Replacement Instructions	23
Calls for Service.....	23
LIMITED WARRANTY	24
TROUBLESHOOTING GUIDE.....	25
GUIDE TO INDICATOR LIGHTS AND SOUNDS	27
SPECIFICATIONS.....	29
IN-SERVICE FORM	33

APPLICABLE MODEL(S)

This manual is for the following product numbers and serial numbers:

PRODUCT NUMBER	DESCRIPTION	EFFECTIVE S/N
464-002-XX	464 Exam/Procedure Chair, Swivel Base, 100-240V	190624-XXXXXX to Current
463-002-XX	463 Exam Chair, Swivel Base, 100-240V	190624-XXXXXX to Current

REVISION HISTORY TABLE

DATE	REVISION	CO#/ECO#	DETAILS
04/01/2019	A	CO-0232	Initial release
06/24/2019	B	CO-0258	Revised Set-up Instructions
1/24/2022	C	CO-0368	Updated front cover image, changed In-service form to checklist, updated logo, and other minor text edits. Added warning to controls section. Added QR code.

User Manual

MTI 464 & 463

GENERAL INFORMATION

Purpose of this Manual

This manual covers the safety, installation, instructions for use, and maintenance of this equipment. It is intended that it be used by the service professional installing the equipment and by the medical professionals who have been trained to use it. No repair information is included with this manual because no operator repairs are intended.

Intended Use

This equipment is intended to be used as a chair/table for general examinations and procedures performed by a qualified medical professional in a medical office environment.

Training Options

Training is based on the intended use of the equipment and the proficiency of the medical professionals who will be using the equipment. Training is accomplished either through reading the installation and operation sections of the user manual or through remote training provided by MTI Technical Support. To request training, contact MTI.

PRODUCT INFORMATION

Date of Purchase: _____

Model Number: _____

Serial Number: _____

(see "EQUIPMENT FEATURES" on page 10 for location of serial number label)

Symbols and Warnings

This manual contains Notices, Cautions, and Warnings, which are meant to call attention to particular parts or sections of this manual. Since safety is of our utmost concern, this manual should be read by all personnel who will be operating this medical equipment.

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WARNINGS/AVERTISSEMENTS

Are used to call attention to condition, practice, or procedure, which could cause personal injury if not followed correctly. Please become familiar with all warnings and observe them at all times.

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Are used to call attention to a condition, practice, or procedure which could cause product damage if not used correctly.

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NOTICES/AVIS

Are used to call attention to helpful information regarding a condition, practice, or procedure.

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Safety Instructions

WARNING/AVERTISSEMENT

Read this manual before installation and use.

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WARNING/AVERTISSEMENT

No modification of this equipment is allowed.

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WARNING/AVERTISSEMENT

Do not operate equipment in the presence of flammable anesthetics.

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WARNING/AVERTISSEMENT

Do not leave the equipment unattended during use of programmable buttons (Home/P1-P4) or manual actuation of a function.

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WARNING/AVERTISSEMENT

Ensure equipment, patient, and other objects are in a safe condition before operating the equipment.

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WARNING/AVERTISSEMENT

Verify patient arms, headrest and other adjustable equipment features and accessories are locked and in a safe and secure position before use.

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WARNING/AVERTISSEMENT

Ensure controls, cords, and other parts of the equipment do not create a tripping or other safety hazard for operator or patient.

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WARNING/AVERTISSEMENT

Activate the Safety Lockout setting or the power button (only on select models) to prevent unintentional equipment movements caused by electromagnetic interference, radio frequency interference from other equipment or from accidental activation of the equipment controls. Failure to do so may result in personal injury. Activating the Safety Lockout setting or the power button (only on select models) will not prevent non-powered motion.

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WARNING/AVERTISSEMENT

Exceeding the specified load limit of any equipment component may result in personal injury or property damage. See "Specifications" on page 29.

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WARNING/AVERTISSEMENT

Backrest, seat, footrest, armrest, headrest, and other parts of the equipment should only be used for their intended purpose. MTI is not responsible for use of components outside of their intended use.

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WARNING/AVERTISSEMENT

This product can expose you to chemicals including nickel, which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov and see www.mti.net/faqs for details on California Proposition 65.

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WARNING/AVERTISSEMENT

The magnet in the footrest may temporarily affect the function or programming of some implantable pacemakers or defibrillators. Avoid positioning the torso of patients who have these types of implanted devices near the center of the footrest.

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Electromagnetic Interference

This equipment is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this equipment, do one or more of the following:

1. Ensure other device is operating within its specifications.
2. Plug equipment into isolated circuit.
3. Remove interfering device from room.
4. Increase separation between equipment and interfering device.
5. Contact MTI if electromagnetic interference persists.

Electromagnetic Compatibility (EMC)

WARNING/AVERTISSEMENT

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

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WARNING/AVERTISSEMENT

During use, portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the

chair, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

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WARNING/AVERTISSEMENT

Use of this equipment adjacent to other electrical equipment should be avoided since it could result in improper operation. If such use is necessary, verify normal chair operation before use with patients.

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NOTICE/AVIS

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

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User Manual | MTI 464 & 463

This chair has been tested to comply with the requirements of EN 60601-1-2:2014 Ed.4 for electromagnetic compatibility (EMC). The chair was designed to be operated within the electromagnetic environment specified below.

Guidance and manufacturer's declaration - electromagnetic emissions		
Emissions Test	Compliance	Electromagnetic Environment - Guidance
RF emissions CISPR 11	Group 1	The 464 & 463 Procedure Chairs use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	
Harmonic Emissions IEC 6100-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	The 464 & 463 MTI Procedure Chairs are suitable for use in all establishments other than domestic and those directly connected to public low-voltage power supply network that supplies buildings used for domestic purposes.

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE MTI PROCEDURE CHAIRS

The MTI Exam/Procedure Chair is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the MTI Procedure Chair can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the MTI Procedure Chair as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter Watts (W)	Separation distances according to frequency of transmitter meters (m)		
	150 kHz to 80 MHz $d = (1.17)(VP)$	80 MHz to 800 MHz $d = (1.17)(VP)$	800 MHz to 2.5 GHz $d = (2.33)(VP)$
0.01	0.12	0.12	0.23
.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.70	11.70	23.30

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY

The MTI Exam/Procedure Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the MTI Exam/Procedure Chair should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Conducted RF IEC 61000-4-6	3V rms 150kHz to 80 MHz	3V	<p>Portable and mobile RF communications equipment should be used no closer to any part of the MTI Procedure Chair, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = (1.17)(\sqrt{P}) \text{ 150 kHz to 80 MHz}$ $d = (1.17)(\sqrt{P}) \text{ 80 MHz to 800 MHz}$ $d = (2.33)(\sqrt{P}) \text{ 800 MHz to 2.5 GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3V/m 80MHz to 2.7 Ghz	3V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^aNOTICE: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the MTI Procedure Chair is used exceeds the applicable RF compliance level above, the MTI Procedure Chair should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the MTI Procedure Chair.

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^bOver the frequency range 150kHz to 80MHz, field strengths should be less than 3 V/m.

GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC IMMUNITY			
The MTI Exam/Procedure Chair is intended for use in the electromagnetic environment specified below. The customer or the user of the MTI Exam/Procedure Chair should assure that it is used in such an environment.			
Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 8 kV air	± 8 kV contact ± 2, 4, 8, 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient / burst IEC 61000-4-4	± 0.5, 1, 2 kV for power supply lines ± 1 kV for input/output lines	± 0.5, 1, 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 0.5, 1 kV line(s) to line(s) ± 0.5, 1, 2 kV lines(s) to earth	± 0.5, 1 kV ± 0.5, 1, 2 kV	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 Tested at 230, 220, 127 and 120 VAC.	0% U_T (100% dip in U_T) for 0.5 cycle 0% U_T (100% dip in U_T) for 1 cycle 70% U_T (30% dip in U_T) for 25 cycles 0% U_T (100% dip in U_T) for 250 cycles	0% U_T (100% dip in U_T) for 0.5 cycle 0% U_T (100% dip in U_T) for 1 cycle 70 % U_T (30 % dip in U_T) for 25 cycles 0% U_T (100% dip in U_T) for 250 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of the MTI Procedure Chair requires continued operation during power mains interruptions, it is recommended that the MTI Exam/Procedure Chair be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m, 50 / 60 Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Electrical Requirements

WARNING/AVERTISSEMENT

To avoid risk of electrical shock, this equipment must be connected to a supply mains with protective earth.

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WARNING/AVERTISSEMENT

Grounding reliability can only be achieved when the equipment is connected to an equivalent receptacle marked "Hospital Only" or "Hospital Grade".

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WARNING/AVERTISSEMENT

When using endocardial catheters or high frequency surgical/procedure devices, use non-conductive material to insulate the patient from metal portions of the equipment. Failure to comply may result in electric shock or burns to patient.

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WARNING/AVERTISSEMENT

Connecting electrical equipment to multiple socket-outlet effectively leads to creating a medical electronic system and can result in a reduced level of safety.

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WARNING/AVERTISSEMENT

Equipment must be positioned in normal use such that the power cords can be easily accessed to unplug.

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WARNING/AVERTISSEMENT

Equipment must be connected to an appropriate power source when loss of power source would result in an unacceptable risk.

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WARNING/AVERTISSEMENT

Do not use a three to two prong adapter to plug the equipment into a two conductor outlet, as grounding reliability is not maintained.

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WARNING/AVERTISSEMENT

Do not plug this equipment into a low power or auxiliary outlet. This equipment must be connected to a 100-240 Volt AC (VAC), 47-63 Hertz (Hz) outlet. If this equipment is connected to an insufficient power source, it will cause irreparable damage to this device's electronics.

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NOTICE/AVIS

The power cord may be used to simultaneously isolate both poles of the power supply by removing the power cord at the equipment input or wall receptacle.

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Contraindications

None.

Transportation and Storage Conditions

	Ambient Temperature	-10°C to +50°C (+14°F to +122°F)
	Relative Humidity	20-90% (non-condensing)
	Atmospheric Pressure	700 hPa to 1060 hPa (0.69 atm to 1.05 atm)

Disposal of Equipment

At the end of product life, the equipment, accessories, and other consumable goods may have become contaminated due to normal medical use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

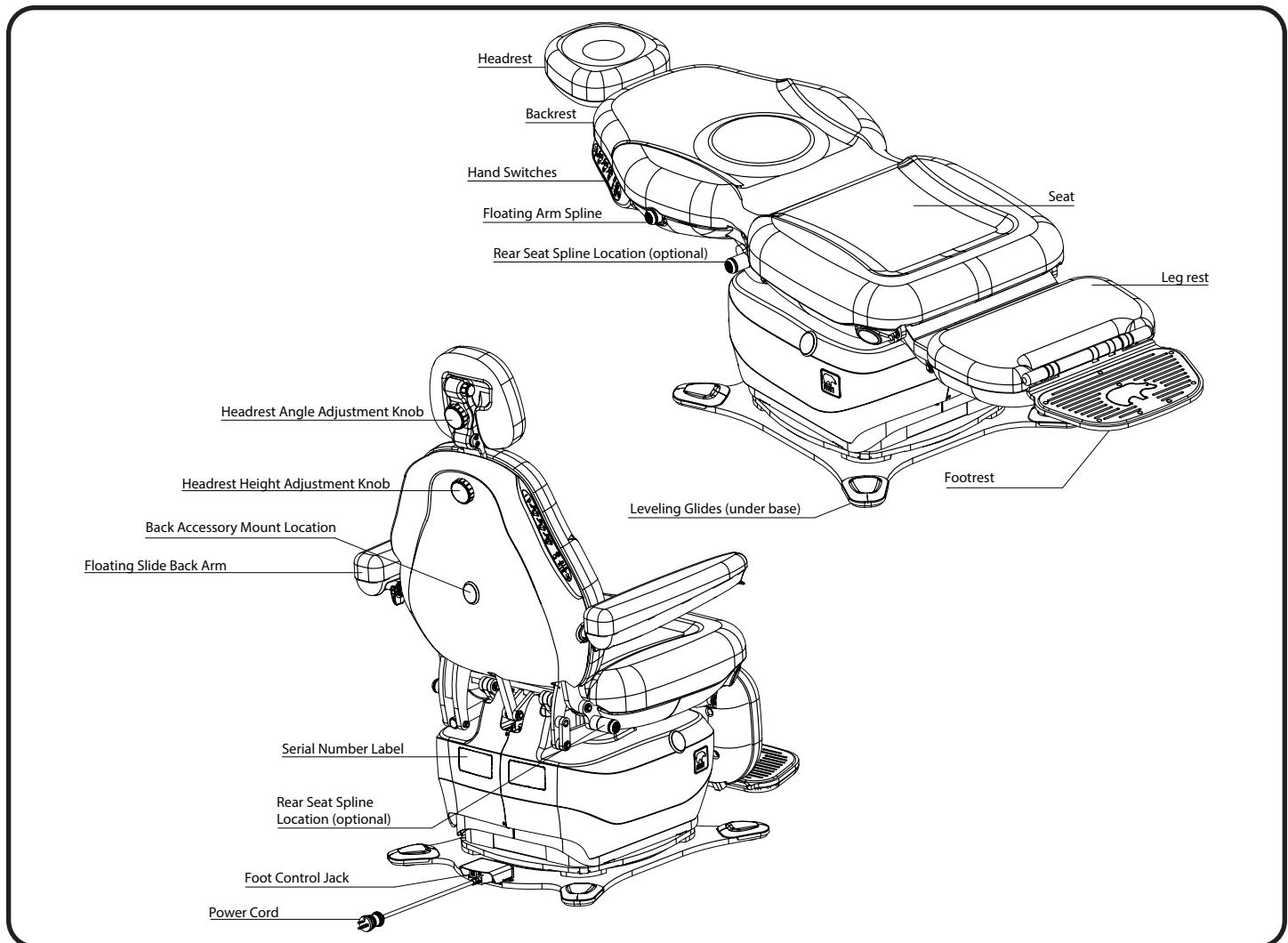
EQUIPMENT FEATURES

Description of 464/463 Chair

The 464 Series Exam/Procedure Chairs have power lift, back, tilt, foot extension, and footrest functions. The 464 Series Chairs have a 725 lb. (328.8 kg) patient capacity. The 464 Series Chairs are 21" (53.3 cm) wide with an entry height of 18 3/4" (47.6 cm). All 464 Series Chairs are Swivel Base models. The 464 Series Chairs will rise to a height of 37" (94 cm).

The 463 Series Exam Chairs have power lift, back, foot extension, and footrest functions. The 463 Series chairs have a 725 lb (328.8 kg) patient capacity. The 463 Series Chairs are 21" (53.3 cm) wide with an entry height of 18 3/4" (47.6 cm). All 463 Series Chairs are Swivel Base models. The 464 Series Chairs will rise to a height of 37" (94 cm).

464/463 EXAM/PROCEDURE CHAIR

INSTALLATION

WARNING/AVERTISSEMENT

This equipment is heavy. Get help to remove it from the shipping pallet. Use proper lifting procedures when lifting to avoid serious back injury.

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WARNING/AVERTISSEMENT

After equipment is installed, verify programmable positions (Home/P1-P4) are safe and known positions.

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CAUTION/PRÉCAUTION

To move equipment, lift only from the equipment's main structure.

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Required Tools

	Adjustable Wrench		Utility Knife		Flat Head Screw Driver		7/16" Wrench
	Set of Standard US Allen Wrenches (for Accessories)		Phillips Screwdriver				

Explanation of Instructions

Instructions have steps which are designated by a number. Each step may also have sub-steps designated by a letter. For example, an instruction to fasten an accessory may have 4 steps (1-4) and each of those steps may have sub-steps (A-B) to be performed before moving onto the next step.

NOTICE/AVIS

If this product has been exposed to low temperatures, allow adequate time for it to warm to room temperature before putting it into service.

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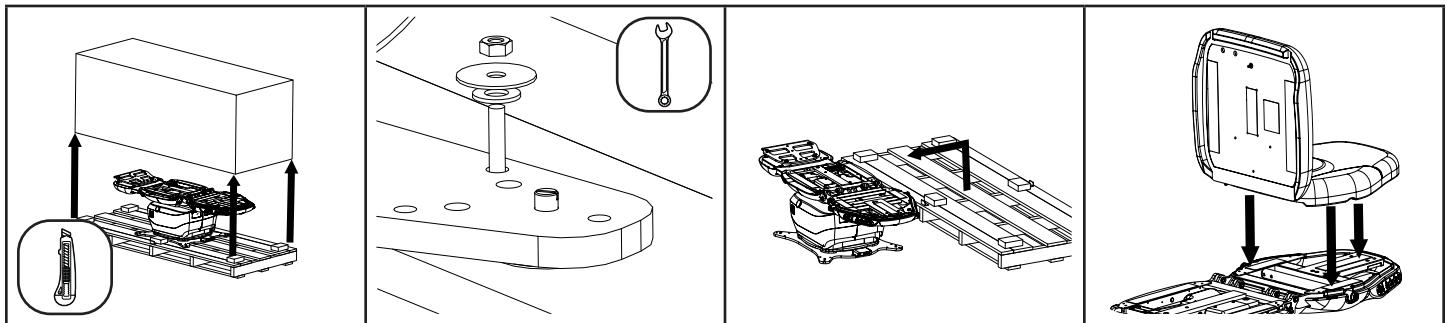
NOTICE/AVIS

Inspect the shipping boxes for visible damage. Record on bill of lading, photograph, report any damage to the carrier immediately, and contact MTI Technical Support. Request an inspection by the carrier if any concealed damage is noted.

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Unpacking and Set-up


1. Remove shipping straps, shipping box, and other boxes from the pallet.

2. Remove (4) shipping bolts with an adjustable or 7/16" end wrench.

3. Slide the chair off the pallet. To move equipment, lift only from the equipment's main structure. Recycle packaging.

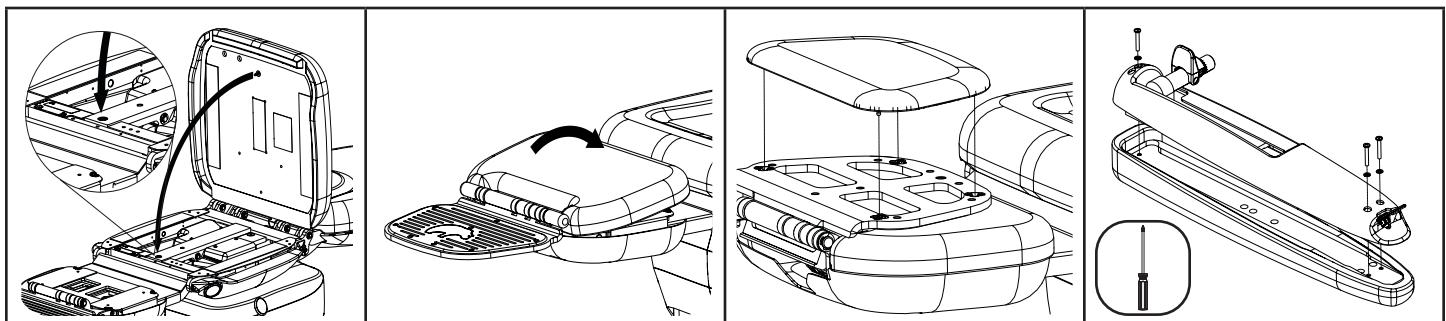4. With the back and seat sections of the chair in the flat position, center the back portion of the seat/back cushion on the back section of the chair while holding up the seat portion of the cushion.


5. Lower the seat portion of the seat/back cushion into place and ensure that the ball stud enters the mating hole in the seat section of the chair.

6. Attach the leg cushion by aligning the ball studs with the 2 receptacles nearest the footrest, then rotate the leg cushion and press until all 4 snap in place.

7. Flip the footrest over. Attach the footrest cushion to the bottom of the footrest by aligning the ball studs to the receptacles and pressing until all 4 snap in place.

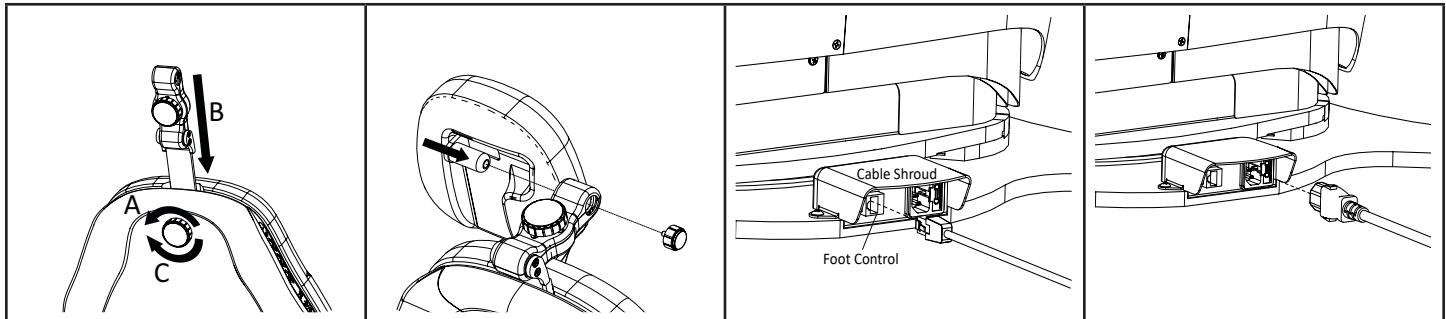8. Using a Phillips screwdriver, attach the arm upholstery to each arm using the provided screws and washers.


9. A. Loosen the knob on the back of the back section of the chair.
B. Insert the headrest lock mechanism into the slotted opening in the top of the backrest.
C. Tighten the knob on the backrest.

10. Loosen the knob on the articulating headrest mechanism and rotate it forward as shown. Remove the knob on the back of the headrest. Insert the headrest into lock mechanism. Reinsert knob, position headrest as desired, and tighten both knobs.

11. If applicable, plug in the foot control at the base of the chair. Note that the foot control jack is located under the rear cable shroud.

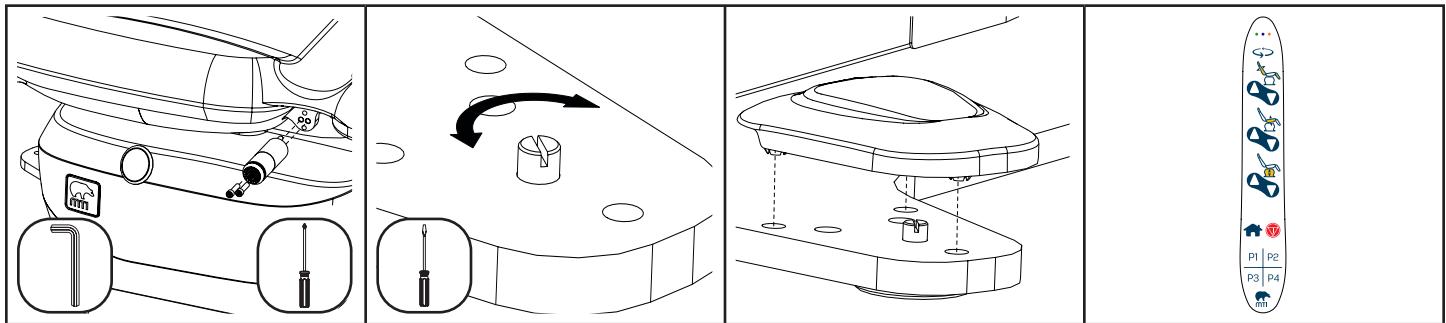12. Insert the power cord at the base of the chair. Press firmly until it bottoms out. Plug the opposite end of the power cord into an appropriately marked receptacle.


13. If applicable, attach any accessories. See the accessory's Instruction Guide for installation instructions.

14. Position the chair in the desired location. Level the chair by adjusting the height of the individual leveling glides with a flat head screwdriver.

15. Install the (4) leveling glide covers over the feet of the baseplate. Align the snap features over the three holes and press down.

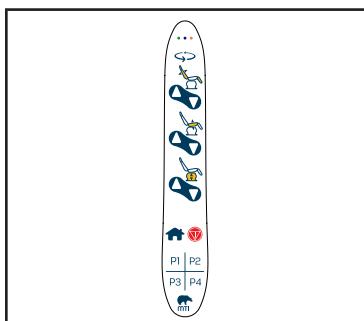16. Verify all functions of the chair are operational. Set the Smart Controller™ Settings for the Home, Program and Swivel Buttons. See "Smart Controller™ Technology" on page 14 for instructions.

17. Save Home and Program Buttons' positions to desired locations. See "Controls" on page 16 for programming instructions.

OPERATION

Operating Conditions

	Ambient Temperature	+10°C to +40°C (+50°F to +104°F)
	Relative Humidity	30% to 75% (non-condensing)
	Atmospheric Pressure	700 hPa to 1060 hPa (0.69 atm to 1.05 atm)

Maximum surface temperature of applied part chair surfaces will not exceed 42°C.

User Manual | MTI 464 & 463

SmartTech™

SmartTech™ is the combination of MTI's Smart Controller™ and Smart Safety™ technology using a centralized computer controller that enhances safety, reduces energy consumption, and provides smoother movement, additional memory options, and more user preferences.

Smart Controller™ Technology

The Smart Controller™ technology is the stack of technology features built into the chair, allowing the chair to move softly and smoothly with soft start/stop technology, a hibernation state to conserve power and a User Program Mode allowing users to change how some of the buttons function.

ComfortSync™

The 464 and 463 chair's ComfortSync™ eliminates the patient's feeling of sliding out of the chair by providing a continual support for the patient's leg and feet while the backrest is laid down. ComfortSync™ provides independent fluid movements of the leg rest and footrest as it follows the natural relationship of the patient's resting leg and foot by automatically extending and maintaining that relationship. The leg rest also extends automatically to compensate for taller patients when the chair lift or tilt is raised and backrest is lowered.

User Program Mode

The User Program Mode allows users to set how the Home, Program and Swivel Buttons function. For user preference these button settings can be changed by entering the User Program Mode to make the setting changes described below.

Home Button and Program Buttons Settings	
Momentary Button Setting	This setting is the factory default setting for the Home Button and Program Buttons. It allows the user to press and release the Home Button or Program Buttons to automatically move the chair to the button's saved position. Press any button, except the same button that was pressed to initiate movement, to stop automatic motion.
Maintain Button Setting	This setting allows the user to press and hold the Home Button or Program Buttons to move the chair to the button's saved position. Release the button to stop movement of the chair.

Swivel Button Settings	
Auto-Lock Swivel Setting	This setting is the factory default Swivel Button setting. Pressing the Swivel Button once will unlock the swivel. The swivel will automatically lock after 30 seconds, or a press of any button (including the Swivel Button) will lock the swivel.
Lock/Unlock Swivel Setting	Pressing the Swivel Button once will unlock the swivel. The swivel will not lock again until any button is pressed (including the Swivel Button). If any button that would result in motion is pressed while the swivel is unlocked, the swivel will automatically lock before moving.

To enter the Smart Controller™ User Program Mode:

1. Press and Hold the **Stop** button for 5 seconds. The chair will beep once and the blue and orange LEDs on the hand switch will flash twice. The Foot Control does not have LEDs, but there are corresponding lights on the chair PCA inside the chair.
2. Press the following sequence of buttons (within 30 seconds from the time the Stop button is released) to enter the Smart Controller™ User Program Mode: **Back Up, Back Up, Back Down, P2, Stop**. Each button press will beep and, if the correct sequence is entered, the chair will sound a low-high tune. If the wrong sequence of buttons is entered, the chair will sound three low beeps and return to normal operation.
3. Within a subsequent 30 seconds, press the following sequences of buttons to change the Home Button and Program Buttons Settings or Swivel Button Setting.

Home Button and Program Buttons Settings	
Momentary Button Setting	Back Up, Back Up, P1
Maintain Button Setting	Back Up, Back Up, P2

Swivel Button Settings	
Auto-Lock Swivel Setting	Swivel, P1
Lock/Unlock Swivel Setting	Swivel, P2

4. The chair will sound a high-low-high tune to accept the setting and the blue LED will flash twice. If the wrong buttons are pressed, the chair will sound three low beeps and the blue LED will not flash.
5. To exit the Smart Controller™ User Program Mode, press the Home Button. The chair will sound a high-low tune and only the green light will be on. The chair will also automatically exit the Smart Controller™ User Program Mode after 30 seconds if no buttons are pressed.

Hibernation State

To conserve power, after 60 minutes of non-use the chair will automatically hibernate. To wake the chair from the Hibernation State, press any button.

Smart Safety™ Technology

The Smart Safety™ technology is the safety features built into the Smart Controller™, making the chair safer and easier to use. The Smart Safety™ technology includes the Collision Avoidance System™, which establishes Safe Zones that will disable a function(s) when the chair encounters a Safe Zone boundary. When the Collision Avoidance System™ detects a Safe Zone boundary, the chair will sound a single tone and disable the chair function(s). To re-enable full chair functionality, move the chair function into the Safe Zone.

Safety Lockout

The Safety Lockout disables all commands to prevent unwanted or inadvertent movements.

To enable the Safety Lockout, press the following sequence of buttons (within 1 second between presses): **Stop, Stop, Back Up, Stop**. If the chair's swivel is unlocked when Safety Lockout is enabled, the swivel will automatically lock and remain locked, even upon disabling the Safety Lockout.

To disable the Safety Lockout, press the following sequence of buttons: **Stop, Stop, Back Down, Stop**.

Shutdown

To shutdown the 464/463 Exam/Procedure Chair, unplug the chair from the outlet or disconnect the power cord from the chair. See "Power Cord Removal Instructions" on page 23

User Manual | MTI 464 & 463

Controls

ITEM	DESCRIPTION	FUNCTION
FOOT CONTROLS AND HAND SWITCHES		
1	Lift Up/Down Buttons	Raises or lowers the chair top by pressing the desired direction.
2	Back Up/Down Buttons	Raises or lowers the backrest in coordination with the leg rest by pressing the desired direction.
3	Tilt Up/Down Buttons (464 Models only)	Decreases or increases the angle of the seat by pressing the desired direction.
4	Stop Button (Emergency Stop Button)	Stops all movement of chair and locks the swivel.
5	Swivel Button	<p>The Swivel Button locks and unlocks the swivel base on the chair. The Swivel Button has two button settings that can be changed by the user.</p> <p>Auto-Lock Swivel Setting: Pressing once will unlock the swivel. The swivel will automatically lock after 30 seconds, or a press of any button (including the Swivel Button) will lock the swivel.</p> <p>Lock/Unlock Swivel Setting: Pressing once will unlock the swivel. The swivel will not lock again until any button is pressed (including the Swivel Button). If the swivel is left unlocked and the chair enters hibernation state (after 1 hour), the swivel will automatically lock.</p>
6	Home Button	<p>The Home Button moves the chair to either a Programmed Home Position or an ADA Compliant Home Position. The Programmed Home Position is user programmable and can be saved in any preferred position.</p> <p>Momentary Button Setting: Press and release the Home Button to move the chair to the Programmed Home Position. Press and release the Home Button twice within 1 second to move the chair to the ADA Compliant Home Position (Lift Down, Tilt Down, Back Up, and Foot Down and Retracted). To save a new Programmed Home Position, position the chair to the desired position then press and hold the Home Button for 10 seconds. Chair will play a high-low-high tune after the Programmed Home Position has been saved.</p> <p>Maintain Button Setting: Press and hold the Home Button to move the chair to the Programmed Home Position. After arriving at Programmed Home, release the Home Button and then press it again within 1 second to move the chair to the ADA Compliant Home position (Lift Down, Tilt Down, Back Up, and Foot Down and Retracted). To save a new Programmed Home Position; position the chair to the desired position, press and release the Stop Button twice, then hold the Home Button for 10 seconds. Chair will play a high-low-high tune after the Programmed Home Position has been saved.</p>
7	Program Buttons (P1, P2, P3, P4)	<p>The Program Buttons move the chair to a user defined Program Position. NOTE: The chair has no pre-existing Program Positions upon arrival and will only beep and not move until Program Positions have been saved.</p> <p>The Program Buttons have two button settings that can be changed by the user.</p> <p>Momentary Button Setting: Press and <u>release</u> to automatically move the chair to the Program Position. To save a Program Position, position the chair to the desired position then press and hold a Program Button for 5 seconds. Chair will play a high-low-high tune after the position has been saved.</p> <p>Maintain Button Setting: Press and <u>hold</u> the button to move the chair to the Program Position. To save a Program Position; position the chair to the desired position, press and release the Stop Button twice, then hold the Program Button for 5 seconds. Chair will play a high-low-high tune after the position has been saved.</p>
8	LED Indicators	Used to provide user feedback on conditions, inputs, and errors. Left to right are green, blue, and orange LEDs. See "GUIDE TO INDICATOR LIGHTS AND SOUNDS" on page 27 for details.

! NOTICE/AVIS

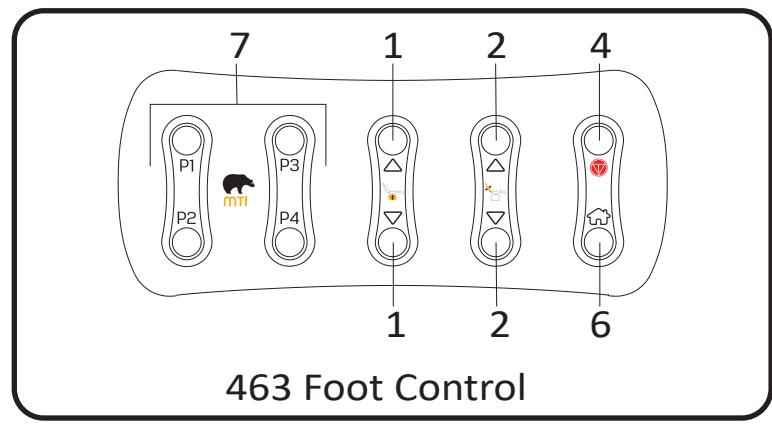
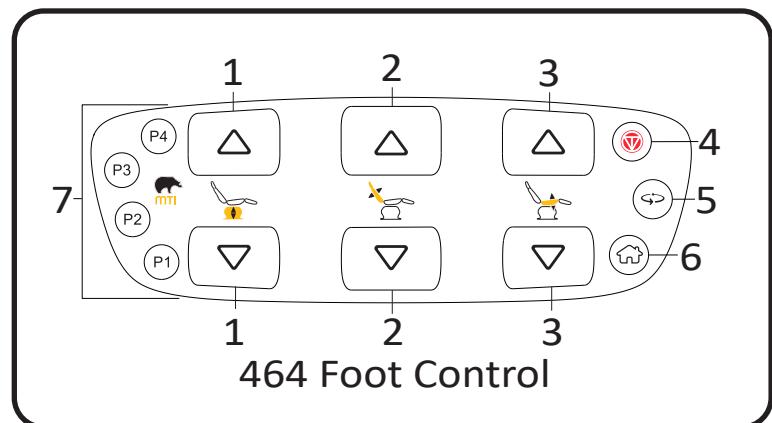
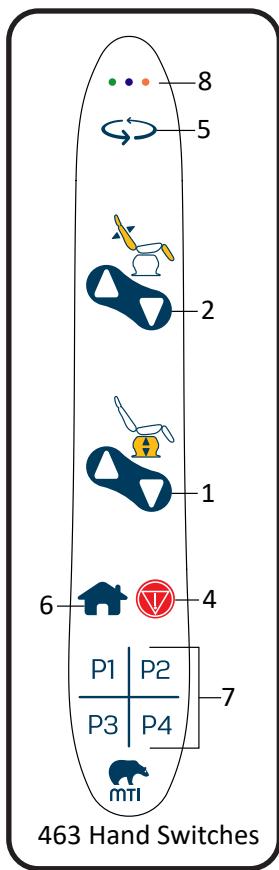
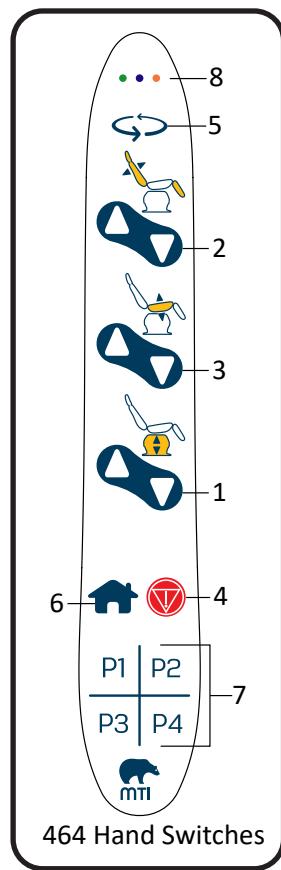
For additional safety, any button will stop all automatic motions except for the same button that was pressed to initiate movement.

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WARNING/AVERTISSEMENT


Do not press combinations of three or more hand switch buttons at one time. Failure to comply may result in involuntary movement and may result in personal injury. If involuntary movement occurs, press the Stop button immediately and contact MTI Technical Support Department.

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Headrest Operation

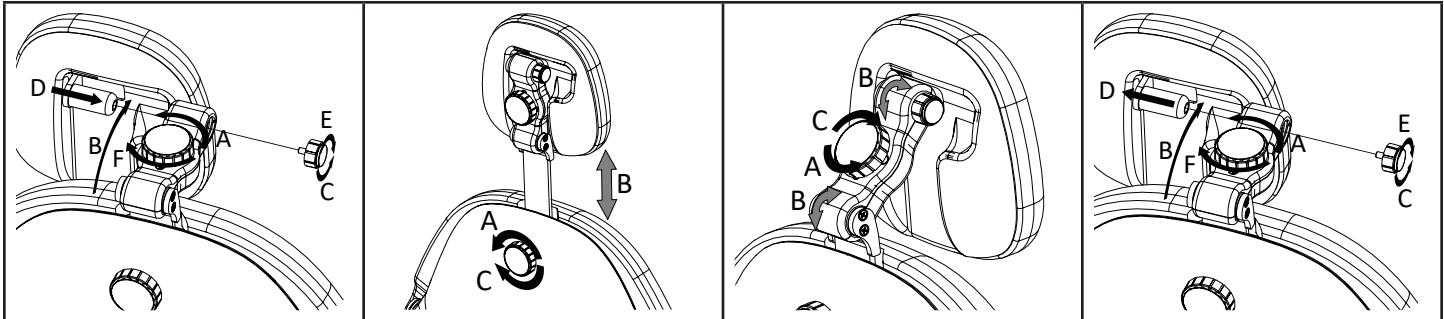Articulating Headrests

1. To attach the headrest.
 - A. Loosen the knob on the articulating headrest lock mechanism.
 - B. Rotate the articulating headrest lock mechanism forward.
 - C. Remove the knob on the back of the headrest (turn counterclockwise).
 - D. Insert the headrest pintle into the articulating headrest lock mechanism.
 - E. Reattach the knob into the end of the headrest pintle (turn clockwise).
 - F. Reposition the headrest to a desired location and tighten the knob on the articulating headrest mechanism.

2. To adjust the height of the headrest:
 - A. To loosen, turn the knob on the back of the chair counterclockwise.
 - B. Adjust the height of the headrest.
 - C. To secure, turn the knob clockwise.

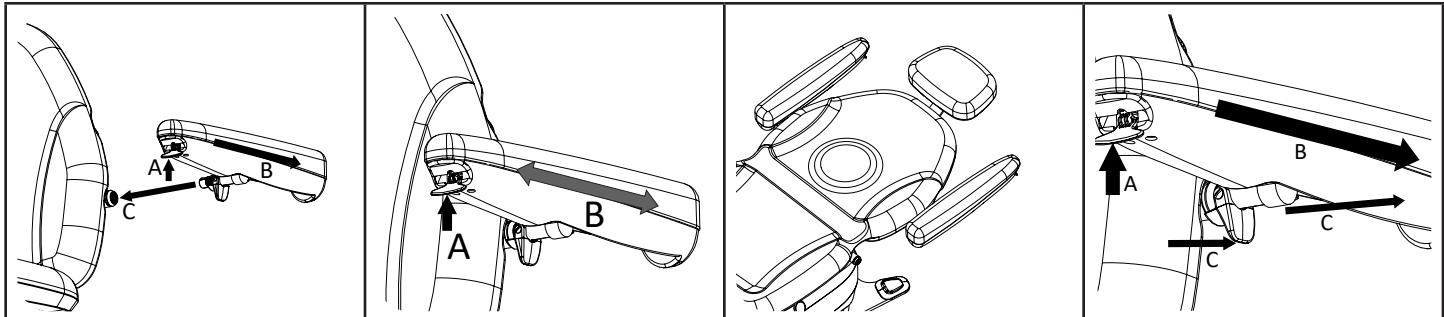
The headrest can be completely removed, if desired, when the knob is loosened.

3. To adjust the angle of the headrest:
 - A. To loosen, turn the knob on the articulating headrest lock mechanism counterclockwise.
 - B. Adjust the position and angle of the headrest.
 - C. To secure, turn the knob clockwise.

4. To remove a headrest:
 - A. Loosen the knob on the articulating headrest lock mechanism.
 - B. Rotate the articulating headrest lock mechanism forward.
 - C. Remove the knob on the back of the headrest (turn counterclockwise).
 - D. Pull the headrest completely out of the headrest lock mechanism.
 - E. Reattach the knob to the end of the headrest pintle to keep it from being lost (turn clockwise).

Armrest Operation

Slide Back Floating Arms

1. To attach the arm:

A. Press the lever on the front of the arm.

B. With one hand, hold the male spline of the arm. With the other hand, move the arm back until it locks into the rear position.

C. Orient the arm to the desired angle and insert the male spline of the arm into the Floating Arm Spline on the side of the backrest.

Confirm that the clip locks into the groove on the Floating Arm Spline.

2. To slide the arm backwards or forwards:

A. Press the lever on the front of the arm.

B. Slide the arm to the desired position until it locks in place.

! *The arm will lock in place in the forward, middle, and rear positions.*

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3. For prone patient positions:

Swap the arms on each side and insert them angled down slightly or parallel to the backrest.

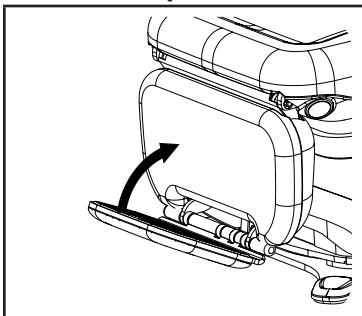
4. To remove the arm:

A. Press the lever on the front of the arm.

B. Slide the arm back until it locks in the rear position.


C. While pulling the lever on the male spline of the arm, pull the arm straight out.

Footrest Operation

1. To fold up the footrest:

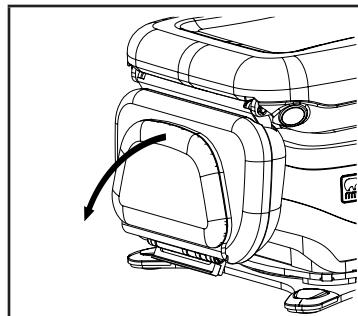Lift up on the footrest until it magnetically latches to the leg rest cushion.

2. To fold the footrest down:

Pull the footrest away from the leg rest cushion and guide it down until it comes to a stop.

⚠ WARNING/AVERTISSEMENT

When the footrest is released from the upright position, ensure equipment, patient, and users are clear of its path to avoid harm.

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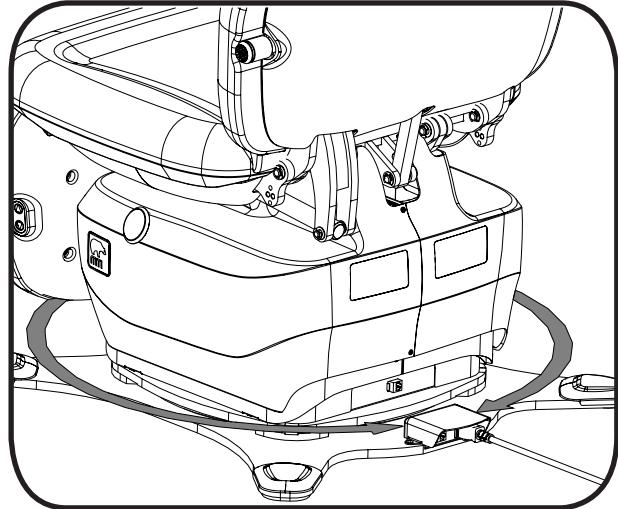
Swivel Base Operation

Use the following instructions to learn how to operate the swivel base and the two user preference settings.

The chair has two Swivel Button settings for user preference. To change the setting, see “To enter the Smart Controller™ User Program Mode:” on page 15. To operate the swivel base, see Swivel Button Settings below.

Auto-Lock Swivel Setting: This setting is the factory default Swivel Button setting. Pressing the Swivel Button once will unlock the swivel. The swivel will automatically lock after 30 seconds, or a press of any button (including the Swivel Button) will lock the swivel.

Lock/Unlock Swivel Setting: Pressing the Swivel Button once will unlock the swivel. The swivel will not lock again until any button is pressed (including the Swivel Button). If any button is pressed that would cause motion, while the swivel is unlocked, the swivel will automatically lock before moving. If the swivel is left unlocked and the chair enters hibernation state (after 1 hour), the swivel will automatically lock.

LIST OF ACCESSORIES

DESCRIPTION	USE	RESTRICTIONS
Headrests		
Deep Oval, Horseshoe, Magnetic, Prosthetic, and Rectangular Articulating Headrest	Provides an articulating surface with various pillow shapes/sizes to support patient's head.	
Lighting Options		
Exam Light, Lumerus LED, Flex Neck, 12-24V DC	Provides lighting during patient procedures.	Requires Adptr Kit, Back Mtd Exam Light w/100-240V Transformer.
Adptr Kit, Back Mtd Exam Light	Provides a mount on the back of the chair for 12-24V Lumerus Exam Light.	Not compatible with Paper Roll Holder.
Other Accessories		
Foot Control, 3 Func (L,B,T)	Provides a foot control to manipulate the position of the chair.	Available only with 464 Model Chair.
Foot Control, 2 Func (L,B,)	Provides a foot control to manipulate the position of the chair.	Available only with 463 Model Chair.
Paper Roll Holder Assy, Adjustable 18" or 21"	Used for mounting a paper roll to the back of the chair.	Not compatible with a back mounted Exam Light.
Power Cord, IEC w/Hospital Grade 90 deg Plug, 3 ft	Provides a 3' power cord with a 90 degree plug.	
Spline Assy, A Size (set of 2)	Provides 2 splines to mount in various places on the chair.	
Strap, Body Restraint	Provides a strap to secure the patient's body.	
Transfer Support/Bed Rail, Flip-up, Ambidextrous	Provides a support for entry and exit into the chair. Also used as a Bed Rail to secure a patient.	Cannot be inserted while an arm is in the Floating Arm Spline. Requires Spline Assy, A Size (Set of 2) mounted in Rear Seat Spline Location.

MAINTENANCE

WARNING/AVERTISSEMENT

There are no user serviceable parts inside. Refer all servicing to a qualified service technician.

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WARNING/AVERTISSEMENT

After equipment is serviced, verify programmable positions (Home/P1-P4) are safe and known positions.

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WARNING/AVERTISSEMENT

The Control of Hazardous Energy (lockout/tagout) 29 CFR §1910.147 procedure must be followed to disable equipment, preventing the release of hazardous energy while performing servicing and maintenance activities.

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WARNING/AVERTISSEMENT

MTI recommends cleaning the equipment frequently. Follow the current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities or other local guidelines for disinfecting.

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CAUTION/PRÉCAUTION

*This equipment may be damaged by solvents and dyes.
Immediately remove any fluids spilled on the equipment.*

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CAUTION/PRÉCAUTION

Contact with sharp or pointed objects can damage the upholstery. Do not place hard surface objects on the upholstery surfaces as temporary indentations or impressions in the

upholstery cover will result. Prolonged contact with a hard or irregular shaped object may make the impression semi-permanent.

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CAUTION/PRÉCAUTION

Any sprayed disinfectant or cleaning solution should be wiped up immediately after application and not let to stand. A small sample area should be tested before any large application of any cleaner/disinfectant.

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CAUTION/PRÉCAUTION

Avoid prolonged direct sunlight exposure to preclude decolorization.

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Disinfection

Follow the current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities or other local guidelines.

Upholstery

MTI understands that many customers prefer to use a pre-mixed disinfecting solution in a spray or wipe. It is MTI's recommendation not to use sprays because over time the overspray will cause damage to the surfaces if not immediately wiped up. See "MTI Suggested Disinfectants Policy" for more details on disinfecting solutions which have been tested on MTI's products. It is available for download at www.mti.net/DisinfectantsPolicy. In the U.S. & Canada, call

(877) 875-4990, or International call (801) 875-4990, or email technical.support@mti.net to request a copy.

Cleaning

Upholstery should be cleaned weekly with a mild liquid soap and water mixture. Afterwards, wipe upholstery with clean damp cloth.

Painted and Plastic Surfaces

All painted metal parts have been electrostatically painted and the paint is baked on for durability. Weekly, those surfaces should be wiped down with a quality grade cleaner and a soft cloth or a damp cloth with mild soap.

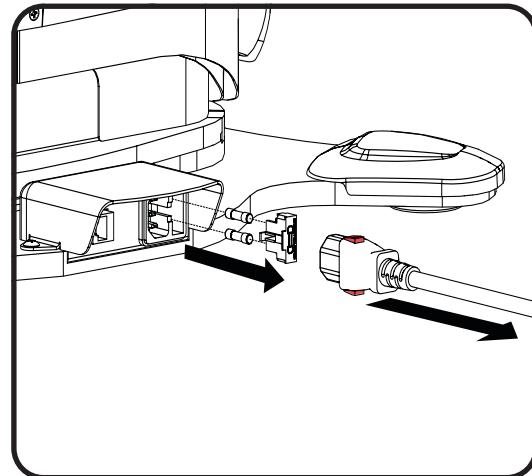
Figure 1

Preventative Maintenance

Periodically inspect the following areas:

1. All fasteners and pins should be in place and tightened securely.
2. Power cord should be free of cuts or other visible damage.
3. All mechanical functions should operate properly.

Have an authorized service technician inspect your equipment once a year.

Power Cord Removal Instructions

1. Pull back on the lower red tab of the power cord. See "Unpacking and Set-up" on page 12 for installation instructions.

Fuse Replacement Instructions

1. See Figure 1. Pull back on the red tabs of the power cord at the base of the chair to remove the power cord.
2. Using a flat head screwdriver, open the fuse drawer in the power cord jack.
3. Remove the burnt fuse(s).
4. Insert new fuse(s) into the fuse drawer with same size fuse (5mm x 20mm 4 Amp 250V Time-Delay Glass tube type) removed in step 3 (See Figure 1).
5. Reinsert the fuse drawer. Ensure it is completely inserted.
6. Reinsert the power cord into the base of the chair.

Calls for Service

Equipment's model and serial number information are required when calling for service.

Serial number is indicated by the **SN** symbol located on the serial number tag.

If service is required, contact MTI.

U.S. & Canada (877) 875-4990

International (801) 875-4990

Fax (801) 952-0548

technical.support@mti.net

www.mti.net

LIMITED WARRANTY

Medical Technology Industries, Inc. (hereinafter referred to as MTI), shall repair or replace products of its manufacture, which prove to be defective in material and/or workmanship for a period of twelve (12) months from the date of shipment to the customer (the "Warranty Period"). MTI will, at its option, provide parts to the customer or repair the defective part at MTI's factory or authorized repair facility. Except as it may otherwise specifically agree in writing, MTI shall not be liable for transportation or labor charges for repairs or adjustments or other work done on any of its products by MTI's authorized dealers or service organizations.

Customer shall notify MTI of any alleged claim or defect within fourteen (14) days from the date customer discovers, or upon reasonable inspection should have discovered, such alleged claim or defect (but in any event before the expiration of the Warranty Period). All repair parts are warranted for Ninety (90) calendar days from the date of shipment to the customer. Warranty is non-cumulative.

This warranty does not apply to any part or product which upon examination by MTI or its authorized agents indicates the product has been repaired or altered in any way by persons other than MTI or its authorized agents, has been misused, abused or used in a manner contrary to any instructions issued by MTI. Any parts or products claimed under this warranty must be properly packaged by customer and returned to MTI's factory with no liability to MTI for the parts, products or transportation charges thereon.

In no event, shall MTI have any liability for consequential, indirect, incidental, special, exemplary, punitive or enhanced damages, lost profits or diminution in value arising out of or relating to a product defect or the repair or replacement of any products manufactured or sold by it.

THE FOREGOING LIMITED WARRANTY IS MADE IN LIEU OF ALL WARRANTIES EXPRESSED OR IMPLIED, AND MTI MAKES NO OTHER WARRANTY WHATSOEVER WITH RESPECT TO THE PRODUCTS INCLUDING BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

See "GUIDE TO INDICATOR LIGHTS AND SOUNDS" on page 27 for additional solutions. For more information on this equipment, including circuit diagrams, parts and service manual (Item No. 130-2725), and other descriptions, please contact MTI.

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#	SYMPTOMS	PROBLEM CAUSE	SOLUTION
1	Chair will not function. Hand switch lights are off. No green light visible on power supply through back of chair.	Chair power cord is not securely connected.	Verify power cord is securely connected at both ends.
		Facility power or receptacle problem. Facility power circuit breaker problem.	Check facility's power receptacle with a known working device of same voltage specifications. Also check facility circuit breaker to verify it is not tripped if no power to known working device.
		Facility supply voltage is incorrect voltage.	Verify voltage is per specifications on chair serial number/product tag.
		Fuses have blown.	Verify fuses are not burnt. Replace fuse(s) if necessary. See "Fuse Replacement Instructions" on page 23.
		Chair Power Supply is faulty.	Contact Technical Support.
2	Chair will not function. Hand switch lights are off. Green light visible on power supply through back of chair.	Faulty hand switches	If possible, test a foot control from similar equipment. Otherwise, contact Technical Support.
		Chair PCA isn't functioning correctly.	Contact Technical Support.
3	Lift, Back, or Tilt functions will not function. Or foot extension/footrest motion does not function.	An actuator is experiencing excessive loads.	Remove or reposition patient, move the chair in opposite direction, then intended direction to continue normal operation.
		An actuator is not functioning or Chair PCA isn't functioning correctly.	Contact Technical Support.
4	The chair will not function. The green light and orange light are lit on the hand switches or Chair PCA.	The Safety Lockout Feature is enabled.	To exit Safety Lockout press Stop, Stop, Back Down, Stop.
5	Swivel will not lock on its own.	The Swivel Button Setting is set to the Swivel Lock/Unlock Setting.	See "Smart Controller™ Technology" on page 14 for instructions to change the Swivel Button Setting.
7	The chair does not move when a Program Button is pressed.	The Program Button Setting is set to the Maintain Button Setting.	See "Smart Controller™ Technology" on page 14 for instructions to change the Program Button Setting.
		The Program Button does not have a saved location.	See "Controls" on page 16 for instructions on saving a Program Button location.

TROUBLESHOOTING GUIDE

NOTICE/AVIS

See "GUIDE TO INDICATOR LIGHTS AND SOUNDS" on page 27 for additional solutions. For more information on this equipment, including circuit diagrams, parts and service manual (Item No. 130-2725), and other descriptions, please contact MTI.

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#	SYMPTOMS	PROBLEM CAUSE	SOLUTION
8	Home Button does not move the chair to the expected location.	The saved location for the Home Button has been set by another user/inadvertently.	Move the chair to the desired location. See "Controls" on page 16 for instructions on saving a Home Button location. For further instructions about the operation of the button, see "Smart Controller™ Technology" on page 14.
9	The chair consistently beeps while moving to a programmed position.	An actuator has gone bad	Contact Technical Support.
10	Chair will not function or a function is erratic.	Chair PCA isn't functioning correctly.	Contact Technical Support.
11	Chair PCA relays turn on/off quickly and excessively ("chatters").	Chair PCA isn't functioning correctly.	Contact Technical Support.
12	Lift function is slower than usual and only goes up about halfway. Chair beeps while moving.	One of the lifting column motors is not functioning or Chair PCA isn't functioning correctly.	Contact Technical Support.
13	Swivel will not lock.	Wiring, actuator, and/or chair PCA is not functioning correctly.	Contact Technical Support.
14	Motor is running after chair stops.	A motor gear has stripped.	Contact Technical Support for motor replacement.

GUIDE TO INDICATOR LIGHTS AND SOUNDS

#	LIGHTS				SOUND	STATUS OR CONDITION		
	GREEN		BLUE	ORANGE				
1		Solid		Off		Off	Start Up Tune	Chair has been plugged in.
2		Slow Flash		Off		Off	None	Chair is in hibernation state to conserve power. To exit hibernation press any button.
3		Solid		Rapid Flash		Off	None	Chair is moving to the Programmed Home or a Program Position or manual button (Lift Up/Down, Tilt Up/Down, Back Up/Down, Foot Up/Down, or Foot In/Out) is being pressed.
4		Solid		Rapid Flash		Rapid Flash	None	Chair is moving to the ADA Compliant Home position.
5		Solid		Off		Off	High Beep when a Program Button is pressed	No Program Position is saved.
6		Solid		Flash Twice		Off	High-Low-High Tune	Program Position or Programmed Home Position has been saved.
7		Solid		Off		Mod. Flash for 10 sec.	High Beep during motion of specific actuator	Actuator/motor Problem (Contact Technical Support).
8		Solid		Off		Flash Twice	High Beep	Chair has been moved to a Safe Zone boundary. (ex. Excessive Trendelenburg or Footrest near Seat Cushion or floor). Move in opposite direction to resume full functionality.
9		Solid		Off		Flash Twice	High Beep	Excessive load on a motor. Remove or shift excess load and move in the opposite direction momentarily to resume. Actuator/motor reaches end of stroke prematurely. NOTE: If beeping occurs frequently, contact Technical Support.
10		Solid		Off		Rapid Flash	None	Swivel has been unlocked.
11		Solid		Off		Solid	2 High Beeps (upon activation)	Safety Lockout feature is enabled. (To exit press Stop, Stop, Back Down, Stop) Orange LED will flash twice each time button is pressed while Safety Lockout feature is enabled.
12		Solid		Off		Slow Flash	High Beep with every button press	No actuators respond to motion commands. Chair ID Mismatch (Contact Technical Support).
13		Mod. Flash		Mod. Flash		Mod. Flash	High Beep with every button press	No actuators respond to motion commands. Chair ID Cable Invalid (Contact Technical Support).
14		Off		Off		Off	None	Loss of power (chair was unplugged).
15		Solid		Flash Twice		Flash Twice	Low Beep	Stop Button has been pressed for 5 seconds awaiting sequence to enter Smart Controller™ User Program Mode or Calibration Mode.

GUIDE TO INDICATOR LIGHTS AND SOUNDS

#	LIGHTS				SOUND	STATUS OR CONDITION	
	GREEN	BLUE	ORANGE				
16		Solid		Flash once		Off	High Beep High Beep with each button press until sequence has been entered for Smart Controller™ User Program Mode or Calibration Mode.
USER PROGRAM MODE							
17		Solid		Solid		Slow Flash Low-High Tune then Low Beep every 10 sec.	Chair has entered Smart Controller™ User Program Mode. Exits automatically after 30 seconds of no activity.
18		Solid		Off		Slow Flash High Beep each button press	Button press while in Smart Controller™ User Program Mode.
19		Solid		Flash Twice		Slow Flash High-Low-High Tune	New setting accepted.
20		Solid		Solid		Slow Flash Three Low Beeps	New setting not accepted.
21		Solid		Off		Off High-Low Tune	Chair has exited Smart Controller™ User Program Mode.
CALIBRATION MODE							
22		Off		Off		Mod. Flash High Beep when an individual motor button is pressed.	One or more actuators is out of calibration. Do not operate the chair. Contact Technical Support.
23		Slow Flash		Slow Flash		Slow Flash Low-High Tune then Low Beep every 10 sec.	Chair is in Calibration Mode. Do not operate the chair. Contact Technical Support.

SPECIFICATIONS

DESCRIPTION	SPECIFICATION
Load Capacity ¹	
Lift	725 lb (328.9 kg)
Back	400 lb (181.4 kg)
Tilt	725 lb (328.9 kg)
Footrest (Stationary Load)	150 lb (68 kg)
Weight of Chair ²	
464 Swivel Base	364 lb (165 kg)
463 Swivel Base	362 lb (164 kg)
Shipping Weight of Chair ³	
464 Swivel Base	428 lb (194 kg)
463 Swivel Base	426 lb (193 kg)
Shipping Weight of Upholstery Kit	
464/463 Seamless/Designer Upholstery Kit	24 lb (10.9 kg)
Table Top Range	
464/463 Swivel Base	18.75-37 in (47.6-94 cm)
Back Section Range	0°± 2.2° (horizontal) to 85°± 2.2°
Table Top Tilt Range	0° ± .6° to 30°± .6° Range is limited when back section is at or lower than 20° below horizontal to avoid collision.
Foot Section Range	0° ± 2.2° (horizontal) to 90°± 2.2°
Foot Extension Range	3 in (7.6 cm) Range is limited depending on position of Lift, Back & Tilt.
Foot Control (optional accessory)	IPX6 Rated
General Dimensions (w/headrest & footrest fully extended)	
464/463 Models	21 in (53.3 cm) wide x 78 in (198.1 cm) long
Power Cord	Plugs: U.S.A. standard 3 prong and IEC 60320-C13 (self-retaining)
Electrical Ratings	100-240VAC (47-63 Hz); 7 Amps
Fuse Replacement	(2) 5mm x 20mm 4 Amp 250V Time-Delay Glass Tube Fuse
Duty Cycle	Intermittent Operation, 10% Duty Cycle Example: (1) minute on (9) minutes off
Classification	Class 1, Type B Applied Part, Ordinary Equipment, Intermittent Operation. Applied parts include: Back, Seat, Leg rest, and Footrest Upholstery, Arms, Headrest, and Footrest.
Listing/Approvals	IEC 60601-1:2005 Ed.3+A1;C1:2014 AAMI ES60601-1:2005+A1 CSA C22.2#60601-1:2014 Ed.3 IEC 60601-1-2:2014 Ed.4

¹Load Capacity will vary depending on placement and distribution of load on the section of the chair.

²Weight may vary due to product variation.

³Weight may vary due to variation in weight of shipping materials.

THIS PAGE IS INTENTIONALLY BLANK

THIS PAGE IS INTENTIONALLY BLANK

THIS PAGE IS INTENTIONALLY BLANK

In-service Checklist for G2 Products with SmartTech™ | MTI

Model Number	Description	Serial Number(s)

The following in-service needs to be performed on this MTI product:

Set Smart Controller™ Settings

Enter the User Program Mode in the Smart Controller™ to change how the Program Buttons and the swivel (if applicable) operate. See "Smart Controller™ Technology" on page 14.

Button Setting

Momentary

Maintained

[Factory default] Momentary Program Buttons: press and release to automatically move chair to the programmed position.

Maintained Program Buttons: press and hold to move chair to the programmed position. Once button is released, chair will stop.

Swivel Setting (if applicable)

Auto-Lock

Lock/Unlock

[Factory default] Auto-Lock Swivel: pressing the Swivel Button will unlock the swivel.

Swivel will automatically lock after 30 seconds or pressing any button will lock the swivel.

Lock/Unlock Swivel: pressing Swivel Button once will unlock the swivel. The swivel will not lock again until any button is pressed.

Saved Programmable Home

Save by positioning chair to desired location. If buttons are set to Momentary: press and hold the Home Button for 10 seconds. If buttons are set to Maintained: Press the Stop button twice and then press and hold the Home Button for 10 seconds. Once saved, audible and visual confirmation will be received.

Reviewed ADA Compliant Home

ADA Compliant Home position is Lift Down, Tilt Down, Back Up, and Foot Down and Retracted. To access: if buttons are set to Momentary, press and release the Home Button twice (within 1 second). If buttons are set to Maintained, press and hold the Home Button twice (within 1 second).

Saved (4) User Program Buttons (P1-P4)

Save by positioning chair to desired location, press and hold the desired program button for 5 seconds. Once saved, audible and visual confirmation will be received.

Reviewed Safety Lockout

Locks out all commands once this state has been enabled. To enter, press the following sequence of buttons: Stop, Stop, Back Up, Stop. Once enabled, audible and visual confirmation will be received.

To exit, press the following sequence of buttons: Stop, Stop, Back Down, Stop. Once disabled, audible and visual confirmation will be received.

Reviewed Controls and Operation sections of the User Manual

If you would like to receive occasional emails regarding MTI promotions and product announcements, fill out the sales form at www.mti.net/contact-us.

Specialized Equipment
for Specialty Healthcare

MTI, Inc.
801-875-4999 • Fax 801-952-0548 • www.MTI.net

THIS PAGE IS INTENTIONALLY BLANK

THIS PAGE IS INTENTIONALLY BLANK

Specialized Equipment
for Specialty Healthcare
Medical Technology Industries

MTI, Inc.

3655 W Ninigret Drive
Salt Lake City, Utah, 84104-6572
800-924-4655 • 801-875-4999 • Fax 801-952-0548
www.mti.net

© MTI All rights reserved. MTI reserves the right to make any product changes without notice. The MTI brand and Bear logo design are registered trademarks of MTI, Inc. in the United States.